# ARNOMED Mundschutz Typ IIR

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### Surgical Face Mask With Elastic Band

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*Surgical Face Mask made of polypropylene non-woven, with elastic rubber band ear loops, flexible nose clip, fiberglass-free, latex-free and soft and comfortable to wear.*

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*Suitable for use in the medical field and protects against the transmission of microorganisms and liquids.*

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*CE classification: Regulation 2017/745 Class 1*

Bakterielle Filtrationsleistung (BFE)  $\geq$  98%

*Bacterial Filtration Efficiency (BFE)  $\geq$  98%*

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*For optimal protection, we recommend the following application for the ARNOMED surgical face mask:*

1. *Take an ARNOMED surgical mask out of the packaging by the outer surface and stretch it open.*
2. *Place the soft ear loops comfortably around your ears, one after the other, while taking care to keep the outside of the face mask turned away from you.*
3. *Bend the nose clip so that it sits tight on the bridge of your nose while comfortably hugging the shape of your face.*
4. *Finally, hold the nose clip and pull the lower end of the mask under your chin to completely cover the area from your chin to your nose.*

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*For optimal protection, we recommend the following application for the ARNOMED surgical face mask:*

- *Make sure that the face mask completely covers your mouth and nose.*
- *Fit the surgical mask closely to the contours of your face. However, the mask should still fit comfortably and not hurt.*
- *When you put on the mask, make sure that you still have enough room to breathe easily.*
- *Do not touch the inside of the mask before and while wearing it so as not to contaminate the surgical mask.*
- *If your mask is soaked due to long or strenuous wear, you should remove and change it.*

**Letzte Änderungen**  
*Last Changes*

21.12.2022