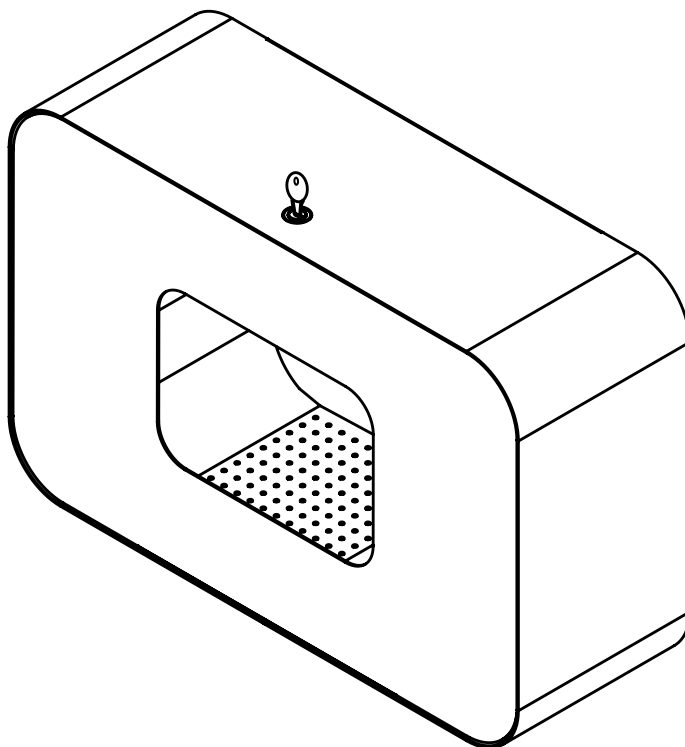
PRAESIDIO®

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The compatibility of filling material and dispenser system must be checked prior to the use of new filling materials, in order to be able to guarantee fault-free functionality. OPHARDT Hygiene assumes no liability for any damages or costs which arise due to the incompatibility of filling material and dispenser systems.

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Please read through carefully these operating instructions prior to initial operation and keep it.

In case of a resale of the apparatus, please pass on these operating instructions to the buyer.

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14. Technische Daten.....19

15. PRAESIDIO® Versionen.....20

1. Indications for the use of the service instructions _21

2. Intended use.....21

3. Safety information.....22

4. Technical Drawing.....23

5. Scope of delivery.....23

6. Mounting of the dispenser.....24

7. Going into operation.....26

8. Change of the dosage.....28

9. Pump removal.....28

10. Maintenance / attendance and maintenance.....29

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1. Indications for the use of the service instructions

The following symbols are used in the instructions

**Safety reference:**

A non-observance may cause damage to persons or defects at the device

2. Intended use

This appliance is suitable only for hand disinfection with low-viscosity disinfecting agents.

It is **not** for use with soaps, high-viscosity products, alco-gels, or abrasive and aggressive agents.

The dispenser is not for free-standing use. Use this appliance indoors only.

Use the appliance for hand disinfection and cleaning, under normal use cycles.

Ambient conditions:

-  • Never use the appliance in potentially explosive areas.
-  • Only use the appliance in a dustless environment.
-  • Pay attention to that the apparatus is only used at a relative air moisture of <90% (not condensing).
-  • Only use this apparatus at room temperature 15 - 30 °C
-  • Ensure that the room size and ventilation are adequate when choosing the installation location.
-  • Please check whether it is necessary to upgrade the available firefighting measures.
-  • Use suitable means, such as ceramic floors, to prevent electrostatic charging of the user.

EN 3. Safety information

3. Safety information

Attention!

OPHARDT Hygiene does not assume any liability for damages in the case of:

- defects at the appliance caused by mechanical influences or overvoltage
- changings at the appliance without express approval of **OPHARDT Hygiene**
- use for other purposes than that mentioned in the application

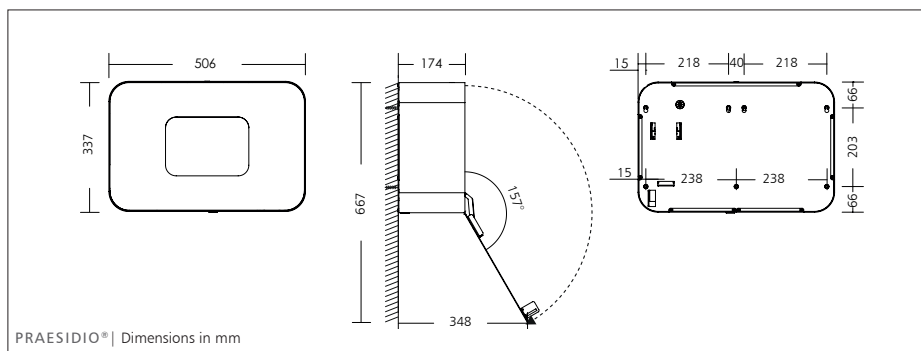
3.1 General safety

- ⚠ • Repairs at the appliance may exclusively be carried out by **OPHARDT Hygiene** employees or persons authorized by **OPHARDT Hygiene**. Important dangers may occur by unexpert repairs. In case of a necessary repair please contact the manufacturer.
- ⚠ • Suspend the appliance on a dry and splash-proof place.
- ⚠ • This appliance is not a toy. Keep and use it out of the reach of children. The appliance should be installed where children cannot reach it.
- ⚠ • Persons with physical, sensory or mental impairment should be supervised when using this appliance.
- ⚠ • The appliance is not suitable for desktop installation
- ⚠ • The appliance is not suitable for the transportation of media.
- ⚠ • The appliance is not suitable for the metering of other than the indicated media.
- ⚠ • Exclusively operate the appliance with leakproof alcalic batteries of the same manufacturer and type and with the same charging condition. Only use batteries with the indicated size type D.
- ⚠ • Do not operate the appliance with rechargeable batteries (accumulators).
- ⚠ • Old batteries may not be mixed with new batteries.
- ⚠ • The battery supply clamps may not be short-circuited with each other.
- ⚠ • Do no longer use the apparatus if batteries have leaked in the appliance.
- ⚠ • Never use force.
- ⚠ • Take the batteries from the appliance prior to cleaning and attendance.
- ⚠ • Only use original parts.

3.2 Safety during operation of the appliance

- ⚠ • Protect your appliance against humidity.
- ⚠ • Do not expose the appliance to thermal radiation and heat.
- ⚠ • Never submerge the appliance in the water.
- ⚠ • Only fill the appliance with the indicated substances in accordance with the intended use.
- ⚠ • There are no dismountable parts with the exception of pump, receptacle, battery cover, batteries, drip tray and outlet pipe
- ⚠ • The appliance is not autoclavable and cannot be cleaned in the dishwasher.
- ⚠ • The apparatus shouldn't be cleaned with water jets or by means of a high-pressure cleaner.
- ⚠ • Please make sure that the medium does not reach the ground.


4. Technical Drawing

5. Scope of Delivery

- Dispenser
- Mounting-Set with 4x plugs and screws
- Batteries (5 mono-cells, type D, 1,5 V)
- Instruction manual
- Key
- Receptacle
- Power supply (optional)

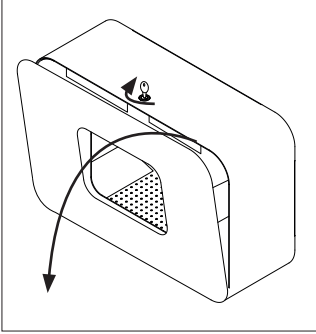EN 6. Mounting of the dispenser


6. Mounting the dispenser

6.1 Wall mounting

During mounting and first use of this PRAESIDIO® appliance, please follow the instructions below in the order given.


The appropriate installation height for safe and comfortable operation is 100–110 cm, measured from the floor to the base of the dispenser.

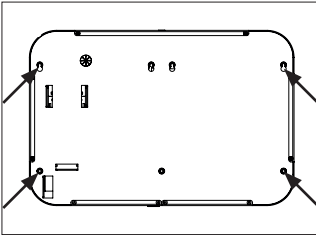Installation of this appliance by two persons is recommended. Open the appliance with the enclosed key and lower the face shield. Before mounting, please remove the clean tray and the receptacle from the appliance. (See sections 7.4 and 7.6).

- The dispenser must be mounted to the fixing apertures provided on a non-sloping wall using at least 4 screws.
- Mount the appliance on the wall using the enclosed fixings or other suitable fixings.
- Check the suitability of the wall before mounting!
- When choosing where to mount the appliance, please consider ease of access for filling!
- The appliance is not suitable for table-top use!
- Do not install the appliance in narrow entrances and passageways!

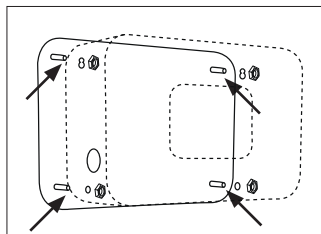EN 6. Mounting of the dispenser


6.2 Column assembly

Place the appliance on the 4 bolts in the mounting wall for the column and secure the appliance with the nuts supplied.

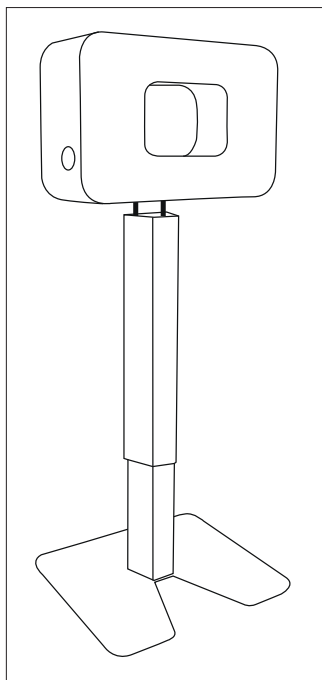
Please ensure that the column is sufficiently stable!

Use only columns that are sufficiently stable and capable of being appropriately fixed.

Accessory: Column (Art.-No. 1421051)

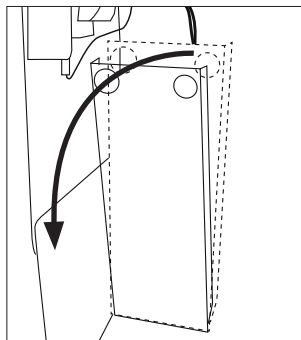EN 7. Going into operation


7. Going into operation

7.1 Insert of the batteries

Open the batterie cover. Please use both holes for pulling down the tray.


Insert the attached batteries into the dispenser.

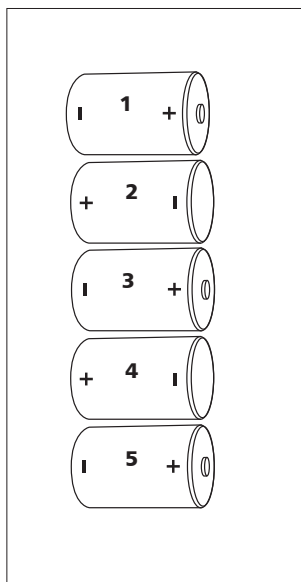Pay attention to the polarity.

Batteries should be removed if it is not used for longer peroids.

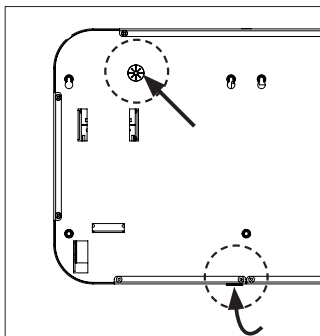EN 7. Going into operation

7.2 PRAESIDIO® with external power supply

Optional: external power supply (Art.-No. 4401040)

AC-Input: 100 -240 V / 0,35 A

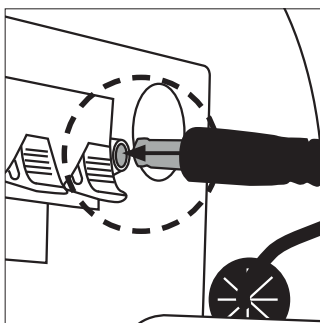
Frequency: 50-60 Herz

Output: 7-0 - 7,5 V, 2,5 A

Special feature: The protective earth is connected with the minus pole.

While mounting the appliance, do not allow the cable to lie across a passageway. Trailing cables are a trip hazard!


The low-voltage plug connector of the power unit is guided through a hole in the rear side or in the base. The batteries are switched off automatically after plugging in the power unit.

Please plug the low-voltage plug connector into the socket provided for this on the inside.

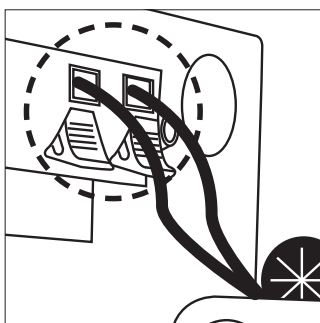- Do not use a power supply that does not meet the above specifications.
Incompatible power supplies can destroy the electronic components.
- Do not connect a 230 V power supply directly to this appliance.

7.3 PRAESIDIO® Door control

A powered door is activated by the door control output.

The connection cable is guided through a prepared hole in the base or in the rear and then clamped at both buttons (red and black).

The polarity is not important.

The output is passive (semiconductors relay contact, closing contact).


During activation of the dispenser, the contact closes for about 0.3 seconds.

Maximum electrical values:

Voltage at the contacts 40V AC / 60 DC,

Short circuit current 100 mA

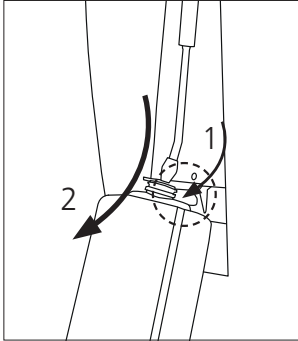
EN 7. Going into operation

7.4 Exchange of receptacle

Pull on the keyhole at the corner of the base and swivel the dispenser forward.

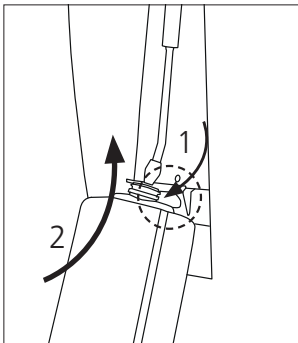Lift up the receptacle from below and remove it.


7.5 Insert of receptacle

To do this, swivel the dispenser and insert the filled receptacle.

The fill level can be checked via the inspection window or by opening the appliance.

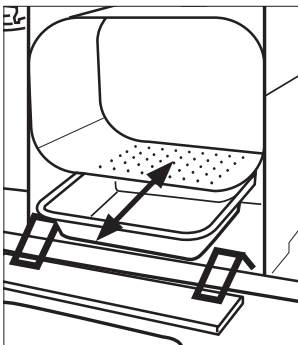
Remove the filling good before transport!

7.6 Remove/Insert clean tray

Remove the clean tray from the guides and re-insert it in the guides after installation/cleaning.

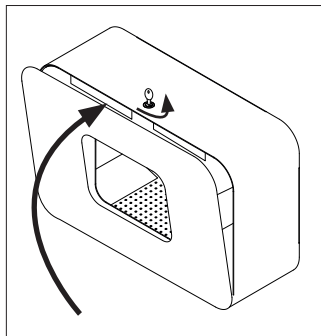EN 7. Going into operation


7.7 Close of appliance

Close the appliance and remove the key. When you close the appliance, it performs a no-load stroke.

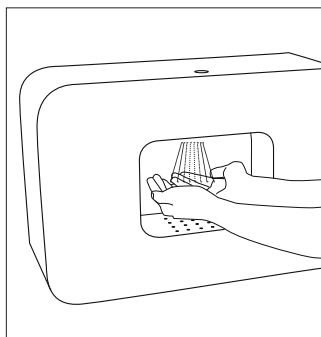
Please wait 10 seconds – do not use the appliance during this time!

7.8 Operation

Place your open hands in the opening and wait until the appliance sprays some disinfecting agent. During commissioning, please press multiple times until the dispenser dispenses hand disinfectant. Rub the agent into your hands for approximately 30 seconds.

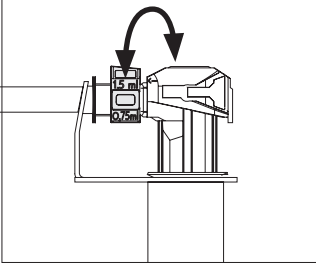
EN 8. Change of the dosage / 9. Pump removal

8. Adjusting the dosage


The dosage of the pump is adjusted to the minimum of approx. 0.75 ml/stroke. If required the dosage can be adjusted in stages by turning the dosage piece (part of the pump) up to approx. 1.5 ml/stroke.

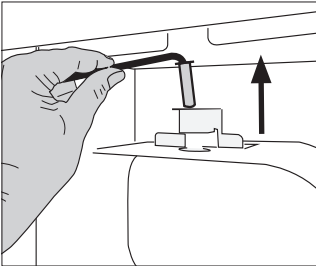For this remove the pump from the dispenser.


9. Pump removal

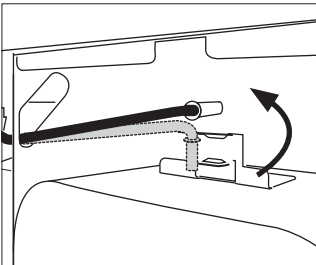Lift the outlet pipe out of the holder.

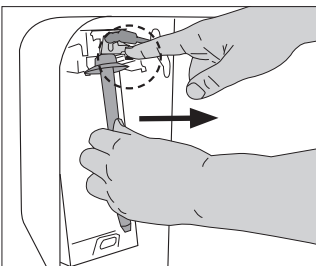Turn the end of the outlet pipe 90° anticlockwise.

Keep the pump block pressed down with the finger and pull the pump out to the front. Simultaneously swivel the top of dispenser unit, to allow the pump to be removed forwards. When inserting the pump, follow the above steps in reverse order.

EN 10. Maintenance / attendance and maintenance

10. Maintenance / attendance and maintenance

10.1 Cleaning and attendance

Please consider that also a high-quality product like this PRAESIDIO®-appliance requires attendance and care. The dispenser itself is maintenance-free with the exception of the battery exchange!

Regularly wipe the interior with a lightly moistened cloth to remove product residue.

Please do not use acidic, caustic (chlorous) purifiers or scouring powder.

 To prevent accidental dispenser operation when cleaning, we recommend removing the holder and pump from the dispenser before cleaning.

If the receptacle is filled up, a non-use during a longer period of time does not lead to an impairment. For that reason exchange the empty receptacle in time. If the suction valve is nevertheless clogged, please only pump hot water through it in order to restore the function.

-  • **The clean tray must be emptied and cleaned each time the receptacle is changed! Rinse the tray out with hot water.**
-  • **Disinfecting agent residues on the pump outlet must be removed regularly!**
-  • **Clean the inside base area only with a slightly damp cloth. Please remove batteries or unplug the appliance from the mains before cleaning. Do not clean near live cables!**
-  • **By no means disassemble the appliance!**
-  • **Please pay attention to that the appliance is not submerged or exposed to water jets.**
-  • **The appliance is not autoclavable and cannot be cleaned in the dishwasher.**
-  • **Please rins the pump with warm water and clean adhesive soap residues from the spray nozzle .**

EN 10. Maintenance / attendance and maintenance**10.2 Exchange of battery****Test the battery level:**

In normal use, the pump does not operate when the batteries run low; instead it emits a short signal tone to indicate that the batteries must be changed.

10.3 In time refill of the medium

For an interruption-free hygienic supply the receptacle has to be exchanged in time.

EN 11. Risks and Troubleshooting

11. Risks and Troubleshooting

 If the appliance malfunctions, please remove batteries or unplug it from the mains.

Characteristic	Possible cause	Solution
1. The appliance seems to function normally but no medium penetrates.	- No receptacle is inserted. - No pump is installed. - Appliance is not closed - Pump is clogged. - Receptacle is empty. - Pump is not touched for operation or lost contact to the medium. - An unsuitable medium has been used.	- Insert full receptacle. - Insert pump into the dispenser. - Close the appliance - Remove pump and clean it. - Replace receptacle. - Carry out the dosage process several times. If it doesn't function, the pump is possibly clogged. - Use another medium. Clean the pump thoroughly before.
2. The appliance doses too much or not enough medium	Dosage adjustment is not adjusted correctly.	Carry out dosage adjustment on the pump
3. The appliance doesn't dose	- Batteries are empty - Power supply has not been connected correctly. - Batteries have not been inserted correctly.	- Insert new batteries. - Check the connections - Pay attention to the polarity

EN 12. Warranty / 13. Disposal

12. Warranty

2 years manufacturer-warranty from the date of delivery.

13. Disposal

Dispose of the packing material into the corresponding recycling plants.

The symbol on the product or its packaging indicates that this product should not be treated as standard household waste, but should be taken to a recycling point for electrical and electronic devices.

Further information may be obtained from your municipality, the municipal disposal companies or the company from where you purchased the product.

13.1 Battery disposal


Batteries should not be disposed of as household waste. As a consumer, you are statutorily obliged to return used batteries. You can dispose of old batteries at domestic collection points, public collection points in your municipality or at any location where batteries of this type are sold.

EN 14. Technical data

14. Technical data

Shroud.....	stainless steel, brushed	Voltage.....	7.5 volt DC
Cover.....	stainless steel, powder coated, white	Power supply (optional).AC-Input: 100 -240 V/0,35 A	
Receptacle.....	synthetic material, removable 1000 ml	Frequenz: 50-60 Herz	
Pump.....	stainless steel, removable	Output: 7,0 - 7,5 V, 2,5 A	
Wall mounting.....	with dowels and screws	Batteries.....	5 mono-cells type ANSI D,1,5 V
		Dimensions in mm.....	B 506 x H 337 x T 174
		Total weight.....	12,75 kg (without batteries)
		Conformity.....	RoHS compliant

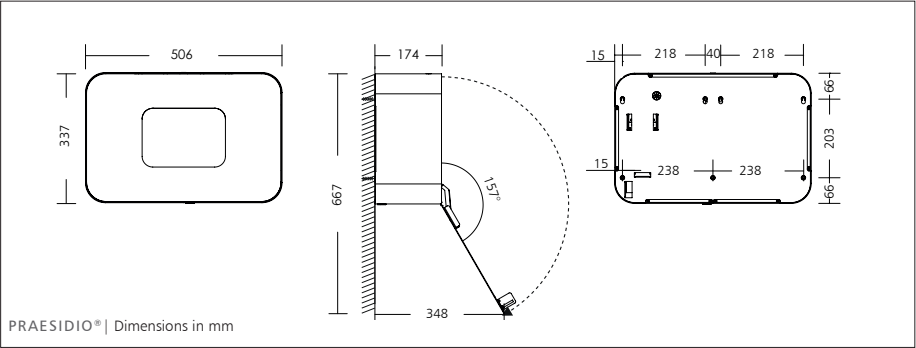CE Machinery directive 2006 / 42 / EC
EMC- directive 2004 / 108 / EC

Technical modifications reserved.
Prior to the use of new filling media tests are required!

EN 15. PRAESIDIO® Versions**15. PRAESIDIO® GT-B3****Intended use**

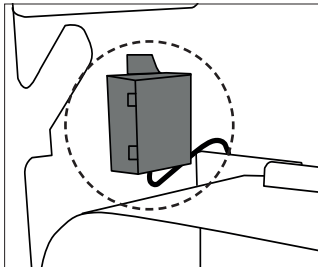
The Bluetooth module may only be used for automatic door opening with the smartdoor TURN T100 door operator.

Installation

Please follow the steps as described in point 6 of these instructions.




Activate these recommended security settings of the TURN smartdoor:

- Motion detector for securing the swivel range of the door in the opening direction (SIO).
- Motion detector for safeguarding the door's swivel range in the closing direction (SIS).
- Emergency stop button, which an immediate stop of the door operator can be triggered before it moves to the predefined position (EMERGENCY opening or EMERGENCY closing).

Going into operation

- Remove the pump with the nozzle from the unit as described in point 9 of these instructions. Make sure that the black plastic housing is accessible.
- Remove the cover by lifting it off. The Bluetooth module is now visible.
- After installing the door opening module, replace the cover of the black plastic housing.
- Reinsert the pump with the nozzle into the unit.
- If the unit is used with the door control, the Bluetooth connection to the door operator must be tested. If the position of the dispenser is changed, the connection must be tested again.
- After activating the dispenser, the door contact closes for 1 second.

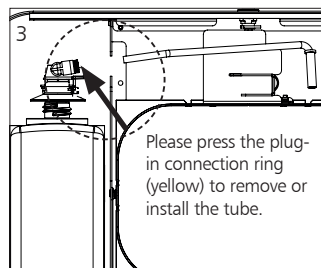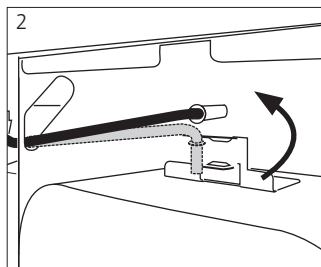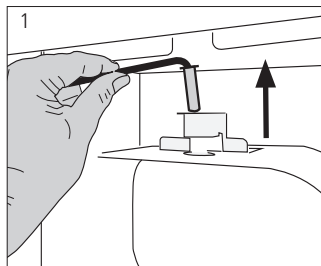15.2 PRAESIDIO® for alco-gel

- To dispense alco-gel, please mount the enclosed nozzle. To do this, pull the showerhead nozzle off the pump (Fig. 1-3) and insert the alco-gel nozzle at the plug connection (yellow), **do not screw it in**.

Intended use

This appliance is suitable **only** for hand disinfection with high-viscosity disinfectants.

- The appliance **is not** suitable for use with low-viscosity disinfectants.
- Use this appliance indoors only.
- Use the appliance for hand disinfection and cleaning, using established hand hygiene standards.

EN

OPHARDT Hygiene-Technik GmbH + Co. KG
Lindenau 27 | 47661 Issum | Germany
Tel.: +49 2835 18-10 | Fax: +49 2835 18-77
www.ophardt.com | E-Mail: customer care@ophardt.com